



# neoResus

The Victorian Newborn Resuscitation Project

---

## Pedi-Cap CO<sub>2</sub> detector

[www.NeoResus.org.au](http://www.NeoResus.org.au)

© 2009

# Background

---

- Clinical methods of endotracheal tube placement have not been systematically evaluated in neonates
- End tidal CO<sub>2</sub> detectors identify oesophageal intubation faster than clinical assessment (Mean 8.1s versus 39.7s)

(Garey, *et al.*, 2008)

# Clinical verification of ETT position

---

## Tracheal intubation is likely if:

- The ETT is visualized passing through the vocal cords
- The heart rate rises above 100/min soon after intubation & commencing positive pressure ventilation
- Breath sounds are auscultated in both axillae
- Condensation is seen on the inside of the endotracheal tube during expiration
- The infant's chest rises & falls with each inflation

(Australian Resuscitation Council, 2006)

# ARC recommendation

---

*“An end tidal CO<sub>2</sub> detector attached to the outside end of the endotracheal tube is recommended for verification of correct tube placement”*

(Australian Resuscitation Council, 2006, Guideline 13.5)

# Benefits of using a Pedi-Cap

- Quick confirmation of correct ETT placement in the trachea
- Easy to use
- Inserts quickly into the circuit
- Inexpensive
- Portable
- Risk management strategy



# How does the Pedi-Cap work?

---

- The Pedi-Cap is a semi-quantitative, non invasive colorimetric end tidal CO<sub>2</sub> (ETCO<sub>2</sub>) detector
- The device starts at a base line colour when minimal CO<sub>2</sub> is present & undergoes gradual colour change as the concentration of exhaled CO<sub>2</sub> increases with each positive pressure breath delivered to the infant
- ETCO<sub>2</sub> is a reflection of ventilation, cardiac output, pulmonary blood flow & metabolism

# The effect of pulmonary perfusion on $\text{ETCO}_2$


---

- If perfusion is adequate,  $\text{ETCO}_2$  represents the partial pressure of  $\text{CO}_2$  in circulating blood. This will be demonstrated breath-to-breath on the Pedi-Cap after successful intubation
- Inadequate cardiac output & decreased pulmonary perfusion (e.g. during cardiac-respiratory arrest) will lead to negligible  $\text{ETCO}_2$  detection as  $\text{CO}_2$  is not being delivered to the lungs

(Garey, *et al.*, 2008)

# Connecting the Pedi-Cap

- The Pedi-Cap is inserted between the outer end of the endotracheal tube and the manual ventilation device (e.g. Neopuff™ or self inflating bag)

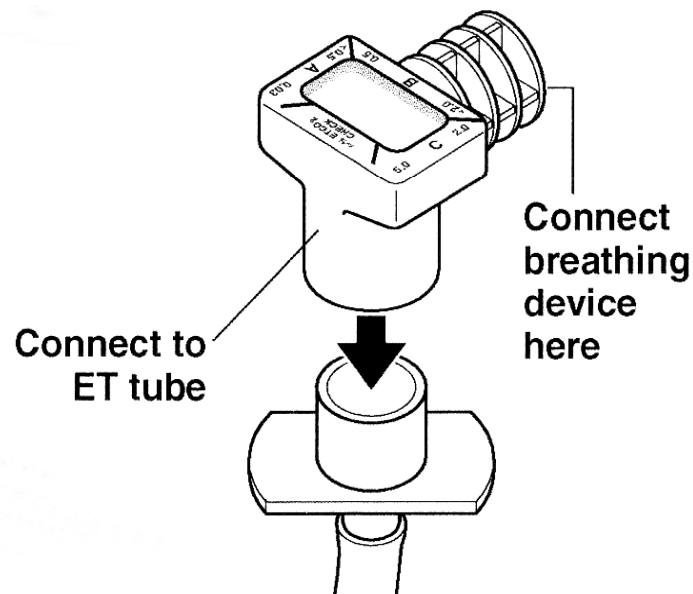


# Interpreting the results


- After 6 effective positive pressure breaths, evaluate the colour of the window on expiration
- Successful tracheal intubation is confirmed if the Pedi-Cap window changes from purple (on inspiration) to yellow (on expiration) with every positive pressure breath delivered to the infant

The following slides describe all the colour changes that may be seen and their meaning

# Successful tracheal intubation

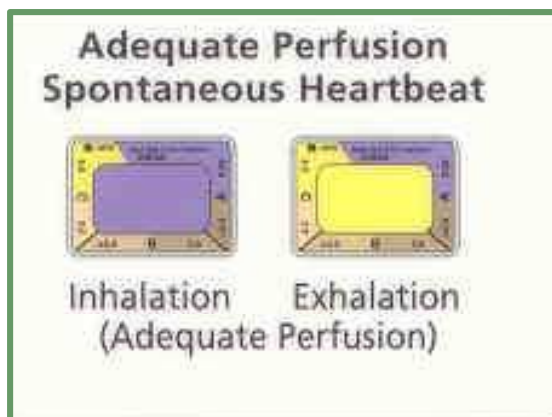


The Pedi-Cap window changes colour on expiration from purple to gold or mustard yellow

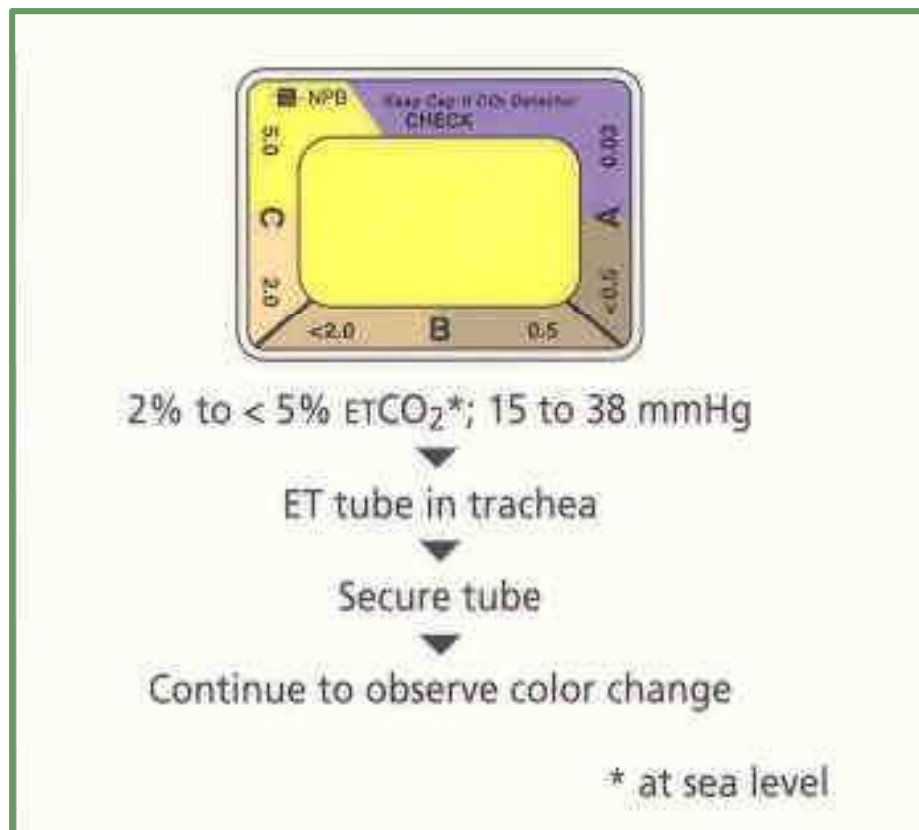


# Poor perfusion or insufficient tidal volume ( $V_T$ ) is being delivered




The Pedi-Cap window changes colour on expiration to light or dark tan if perfusion is poor or insufficient  $V_T$  is being delivered

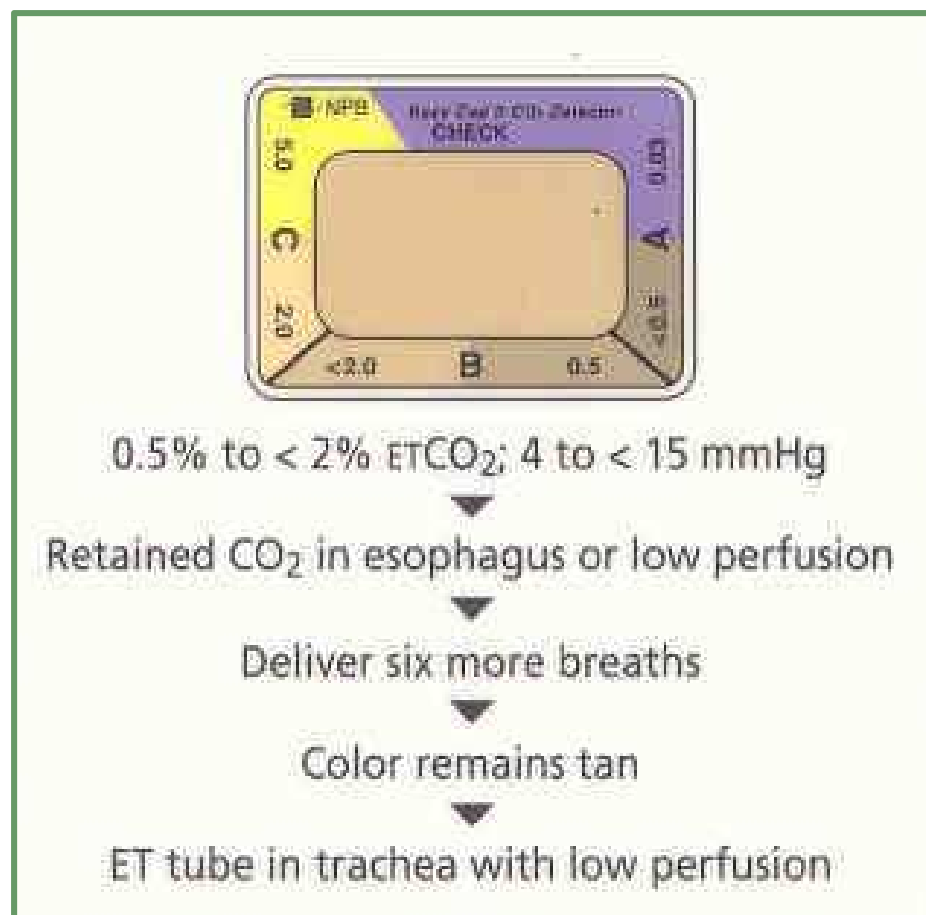

# No perfusion, cardiac arrest or oesophageal intubation

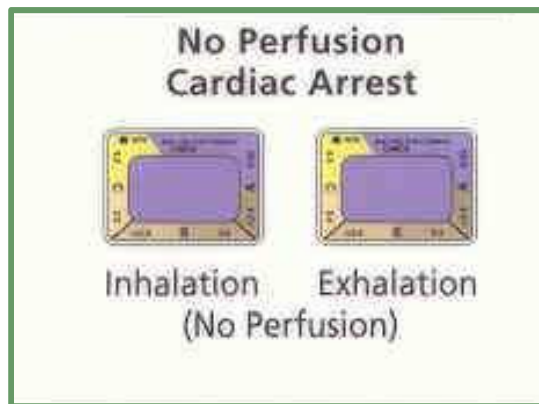


The Pedi-Cap window stays purple or dark grey on expiration if there is no perfusion or the ET tube is in the oesophagus

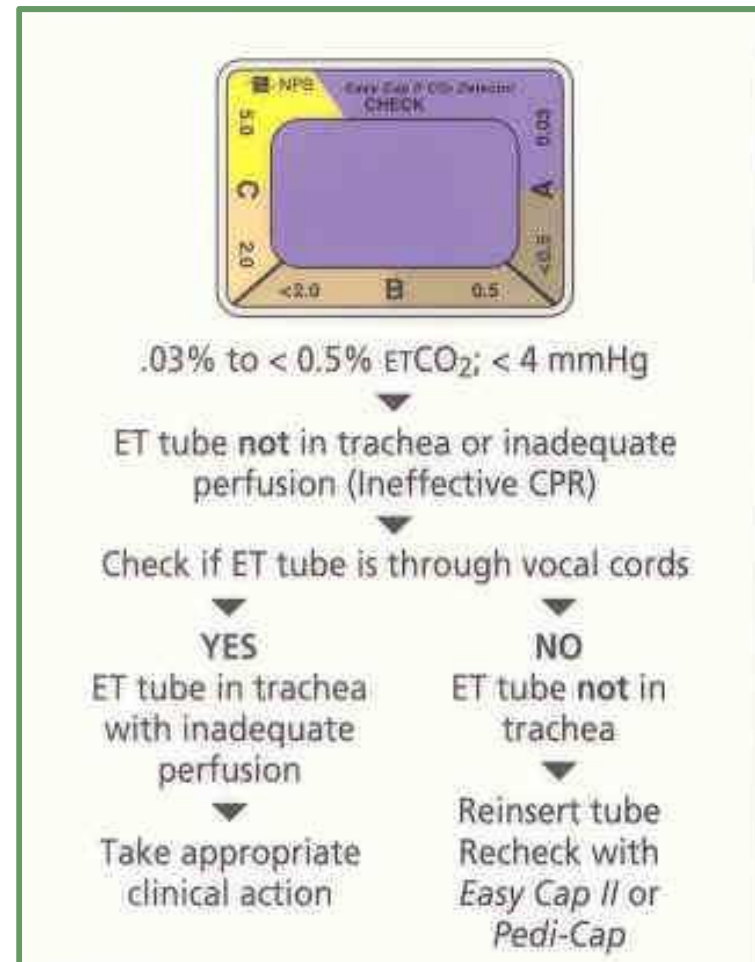

# Damaged Pedi-Cap

## Damaged Pedi-Cap

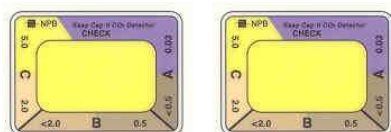

Inhalation      Exhalation

The Pedi-Cap window stays yellow on both inspiration & expiration, indicating a damaged Pedi-Cap

## Reasons for damage

- Contamination with Adrenaline
- Contamination with surfactant
- Exposure to gastric juices
- Prolonged exposure to high humidity

# Caution when using a Pedi-Cap

---

- Misinterpretation of the result is possible if:
  - Insufficient breaths are delivered
  - Insufficient tidal volume is delivered
  - There is significant air leak around the endotracheal tube
  - The infant is in full circulatory arrest

# Management during cardiac arrest

---

- In cardiac arrest, re-establishment of cardiac output and pulmonary perfusion by adequate CRP is necessary to increase end tidal CO<sub>2</sub> to a level detectable by the Pedi-Cap
- Actions:
  - Continue ECM & positive pressure ventilation at 3:1
  - Check that the ETT can be visualized passing through the vocal cords: re-intubate if it is not
  - If the ETT is through the vocal cords, increase the PIP to ensure a sufficient tidal volume is being delivered

# The very low birth weight infant

---

- The Pedi-Cap CO<sub>2</sub> detector is labeled for use in infants > 1kg birth weight
- Research has shown that the tidal volume of a viable (400g) infant is above the tidal volume threshold for the Pedi-Cap device, suggesting that a Pedi-Cap is appropriate for use on any neonate to confirm intubation

(Garey, *et al.*, 2008)

# Limitations of the Pedi-Cap

---

- A positive colour change will occur when the endotracheal tube is in any portion of the respiratory tree, such as the right main bronchus or oropharynx
- A chest X-ray remains the gold standard to confirm correct endotracheal tube position in any infant who requires ongoing ventilation in SCN or NICU after birth

# Conclusion

---

- The Pedi-Cap can quickly verify endotracheal tube placement in the trachea
- It is easy to use
- It is easy to learn to use
- Caution is required in certain situations
- A Pedi-Cap should be standard equipment on newborn resuscitation cots

(Australian Resuscitation Council, 2006)

# Pedi-Cap: Product details

- Weighs < 5g
- Dead space: 3 mL
- Resistance: 2.5 cm H<sub>2</sub>O (+/- 0.5 cm) at 10 L flow
- Single patient use, but can be used intermittently or continuously on an infant for two hours



**Pedi-Cap®**  
Patient size:  
1–15 kg

# Supplier

---

Tyco Healthcare Pty Ltd

Telephone: 1800 252 467

Pedi-Cap™ Pediatric CO<sub>2</sub> Detector

Box of 6

# References

---

- Australian Resuscitation Council, 2006. *Guideline 13.5. Tracheal intubation and ventilation of the newly born infant*. Retrieved March 8, 2009 from <http://www.resus.org.au>
- Garey, D.M., Ward, R., Rich, W., Heldt, G., Leone, T., & Finer, N (2008). Tidal volume threshold for colorimetric carbon dioxide detectors available for use in neonates. *Pediatrics*, (2008), 121, e1524-1527.
- International Liaison Committee on Resuscitation (2006). ILCOR consensus on science with treatment recommendations for paediatric and neonatal patients: Neonatal Resuscitation. *Pediatrics*: 117, (5), e978 -e988.
- Nellcor. (2009). *Pediatric End Tidal CO<sub>2</sub> detector*. Retrieved March 8, 2009 from <http://www.nellcor.com/prod/PRODUCT.ASPX?S1=AIR&S2=CO2&id=176>

# Acknowledgments

---

This presentation is adapted from an original presentation by Johnston, E., Adams, A., & Stewart, M. (2006).

NeoResus gratefully acknowledges and thanks the authors for allowing their presentation to be adapted for this program

# Disclaimer

---

- The author of this presentation has attempted to ensure the information upon which it is based is accurate and up to date. No responsibility is accepted for any inaccuracies or information perceived as misleading.
- The guidelines contained in the presentation do not indicate an exclusive course of action. Variations, taking individual circumstances into account, may be appropriate.
- You are encouraged to consult other sources in order to confirm the information contained, especially drug information.